


# BREAK THE STRICTURE CYCLE WITH OPTILUME



## Engineered for long-term relief An innovative application for a proven technology

Optilume combines simple, effective balloon dilation with localized delivery of a proven antiproliferative, inhibiting recurrence and improving physician and patient satisfaction.

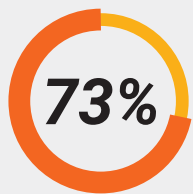


reduction in IPSS\*

\*ROBUST 1 Study at 24 months



increase in Qmax\*

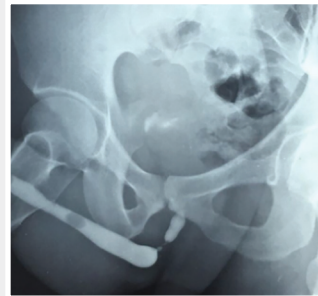


Baseline

### BASELINE

Length of stricture: 1.0 cm

IPSS: 29

Qmax: 4 mL/sec

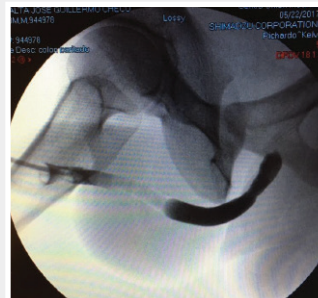

6 months

### 24-MONTH FOLLOW-UP


IPSS: 1

Qmax: 18 mL/sec

## ROBUST I Uroflow and IPSS Results

Traditional endoscopic management of urethral strictures has high retreatment rates and decreasing efficacy with each successive treatment. At 2 years, Optilume demonstrated sustained improvement in efficacy and durability with a single treatment.

### Uroflow Outcomes

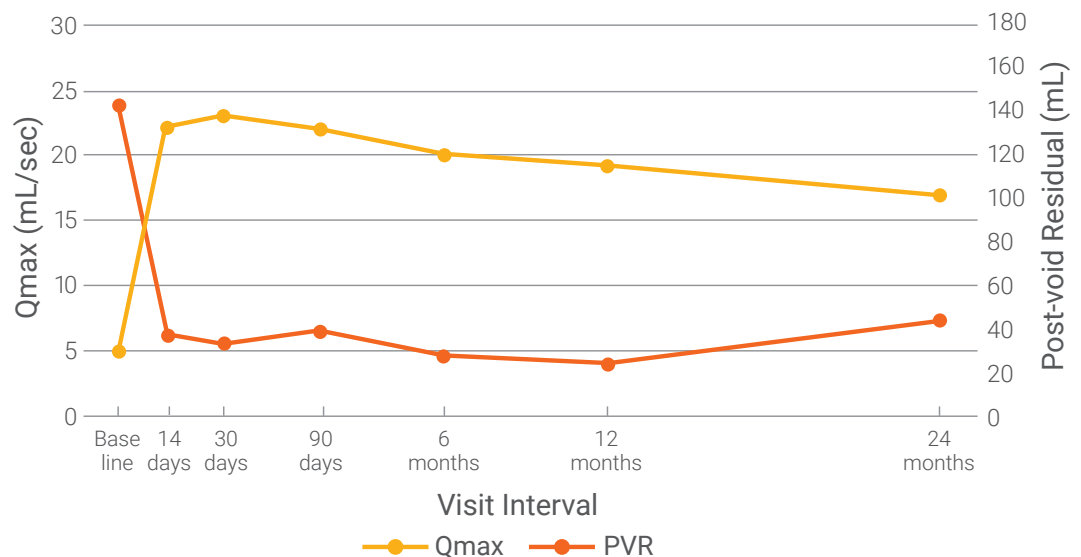

Not for sale in the U.S.

## How it works

### Effective urethral dilation with localized drug therapy

1. Proprietary dynamic-compliant balloon expands at high pressure, expanding the lumen and creating microfissures in the tissue
2. Upon inflation, the coating releases paclitaxel directly into the fissured tissue
3. Paclitaxel is absorbed by the urothelium where it resides for >30 days, preventing cell proliferation and fibrotic scar tissue generation



“

The Urotronic Optilume Balloon Catheter has a significant opportunity to be a game changer for this disease.

– **Gerald H. Jordan**

MD, FACS, FAAP (hon), FRCS (hon)

## Why it works

### Proven effects of circumferentially delivered paclitaxel

The hydrophobic and lipophilic properties of paclitaxel make it the ideal antiproliferative for uptake and residency in urothelial tissue, inhibiting the post-procedure inflammatory response and providing sustained symptom improvement.

The long-term safety (out to five years) of localized, low-dose paclitaxel has been proven in millions of patients in vascular indications.



### PRODUCT ORDER NUMBERS

	BALLOON LENGTH (MM)
INFLATED BALLOON DIAMETER (FR/MM)	30
18.0/6.0	1110-06030
24.0/8.0	1110-08030
30.0/10.0	1110-10030

To incorporate Optilume into your practice, please visit us at [urotronic.com](http://urotronic.com).

CE 1434

Not for sale in the U.S.

UROTRONIC