

ClearView[®]

Uterine Manipulator

OBSTETRICS, GYNECOLOGY & NEONATAL
Providing surgeons with optimal visibility and complete uterine control during laparoscopic procedures.

Single-handed controlled Uterine Manipulator

ClearView[®] Uterine Manipulator, with its 210° range of motion and rotating handle facilitates one-handed control, enabling the surgeon to manipulate the uterus independently.



ClearView[®] Uterine Manipulator

Clear Visibility

ClearView provides 120° anteversion and 90° retroversion as well as lateral movement for clear visibility.

Single-Handed Control

The self-holding rotating control handle allows the surgeon to maneuver the device independent of an assistant.

Patient Positioning

ClearView pivots at the cervical os and allows the patient to remain in the supine position

Single-Use

ClearView is completely disposable.

Packaged with what you need

ClearView is packaged in a sterile tray with everything you need to complete the procedure.

- ▶ The dye-injection port allows for **chromotubation** if needed.

- ▶ **Uterine Control:** the included 10ml syringe inflates the tip balloon.



- ◀ The spacer shortens the tip by 1-cm, giving you **two sizes** in one device.

- ◀ The included **disposable sound-dilator** helps measure and prepare the cervix.

ClearView® Uterine Manipulator


Doing a Laparoscopic Hysterectomy?

ClearView® Total has the same features as ClearView with the addition of a pre-attached occluder balloon and three different sized colpotomy cups. The occluder balloon prevents the loss of pneumoperitoneum once the colpotomy incision is made during laparoscopic hysterectomy.




▲ ClearView® Total, same features as ClearView but with addition of pre-attached occluder balloon and three different sized colpotomy cups.

ClearView® Set Up and Use

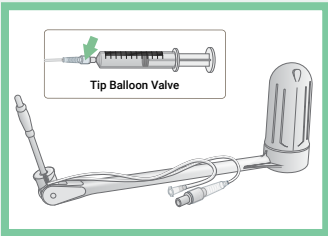


Check Balloon

Use supplied syringe to test the balloon prior to insertion.

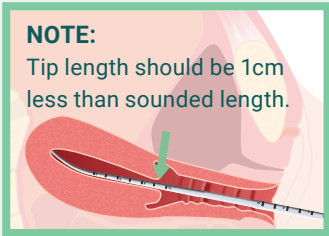


Dilate & Sound Uterus

Sound the uterus. If necessary, dilate the cervix..

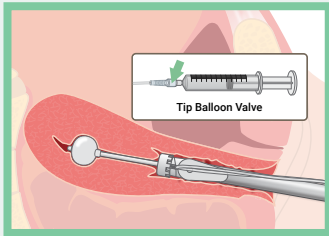

Insert Device & Inflate

Lubricate distal tip and insert into uterus. Slowly inflate.

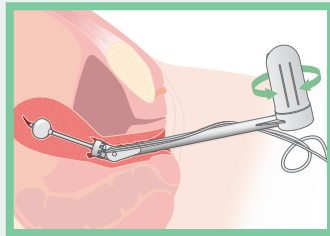

Control Handle

Rotate the handle to manipulate the uterus.

See Instructions for Use for full instructions, warnings, precautions and contraindications

Product Support / Ordering Information

Product Description	Product Code	Box Quantity
ClearView Uterine Manipulator Kit- 7 cm tip	UM-700	5/ea
ClearView Uterine Manipulator Kit- 9 cm tip	UM-900	5/ea
ClearView TOTAL Kit- 7 cm tip	UM-750	5/ea
ClearView TOTAL Kit- 9 cm tip	UM-950	5/ea
Disposable Sound- Dilator (Sterile)	DSD-1000	50/ea

© 2025 Laborie. All rights reserved.

Laborie

USA

T +1 800 522 6743
F +1 802 878 1122

E usmarketing@laborie.com
W laborie.com



Learn more

MKT-01007A 02/25