

Rapid Results in 1 Hour

No Refrigeration & No Reagents

3 Year Shelf Life



# PyloPlus

PyloPlus Rapid Urease Test  
Helicobacter Pylori Test

 **GI Supply**<sup>®</sup>  
Specialty Endoscopic Products


# PyloPlus: The Newest Generation Helicobacter Pylori Test

While there are several methods of detecting H.pylori, biopsy testing using rapid urease is considered the Gold Standard by professionals worldwide.

PyloPlus RUT is the most effective and easy to use Rapid Urease Test on the market today. PyloPlus RUT offers significant improvements that make detecting H.pylori infections in gastric mucosal biopsies faster, easier, less expensive and more accurate than ever before.




Feature	Benefit
No Refrigeration Required	Can be stored at room temperature. No need to keep temperature logs
Rapid Results	Positive in as little as 1 minute negative results in 1 hour
Accurate	99% Sensitivity and 97% Specificity as shown in major US studies comparing PyloPlus to histology and other R.U.T.'s.
Dry Test	No incubation or reagents required
Cost Effective	Low per test cost
Simple And Convenient	CLIA waived
3 Year Shelf Life	Never worry about product dating

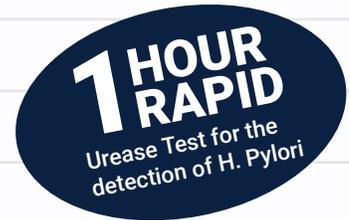


## 5 SIMPLE STEPS:


1. Peel back label

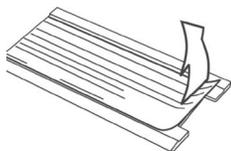

2. Place 1 or more biopsies on test

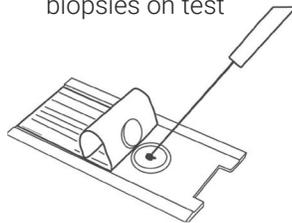

3. Re-seal test

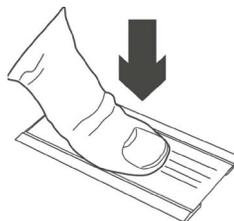

4. Wait up to one hour



5. Read results

Red = positive



Yellow = negative



Product#	Description	Packaging
PYLO-50	PyloPlus Rapid Urease Test	50 Each

Visit [gi-supply.com](http://gi-supply.com) to learn more

GI Supply is an authorized distributor of PyloPlus for ARJ Medical, Inc  
G-1942-01 | © 2021 GI Supply, Inc. | All rights reserved.

