

Intelligent Cath-ID System


Quick User Guide (Goby & Aquarius®)

UROLOGY & UROGYNECOLOGY


Connecting catheters and pump tubing with Laborie Urodynamic Systems




1. Every Laborie catheter and pump tubing box has an RFID tag that contains the part number, number of consumables in the box (catheter & pump tubing) and expiration date.

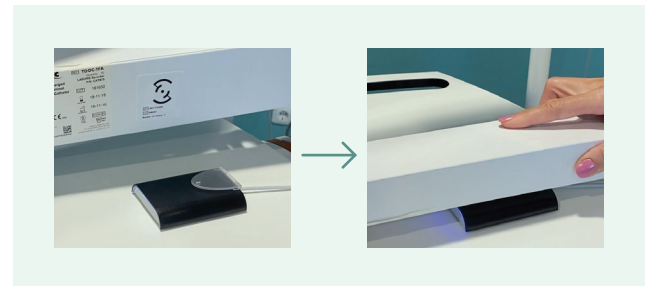

2. Scan the RFID tag on the catheters and pump tubing prior to performing any urodynamics study.

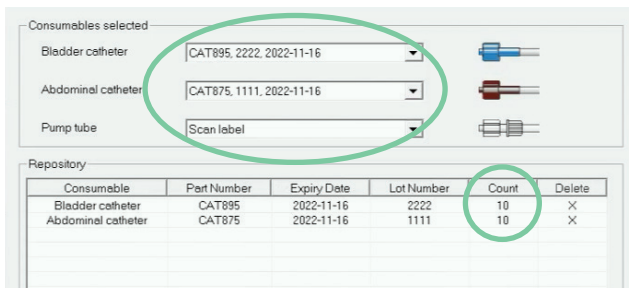


3. The system will automatically detect and add the consumables to a scanned inventory.

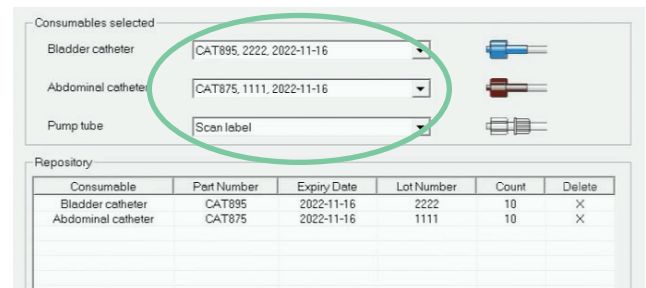


4. To perform a urodynamics study, select the catheters and pump tubing to be used from the drop-down menu.

An RFID tag can only be scanned once.

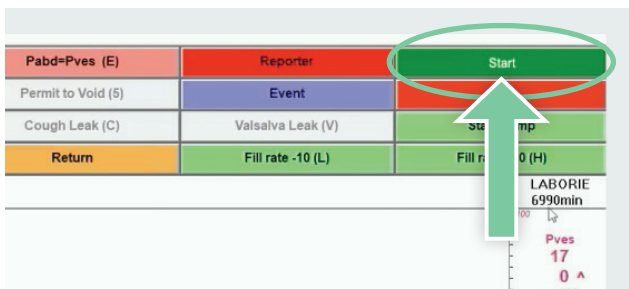


5. Now that the consumables are scanned and selected from the scanned inventory, prepare the catheters for study, and push the START button to activate the urodynamics study.


Once the START button is pushed there is a 2-hour time slot per patient to perform the study.

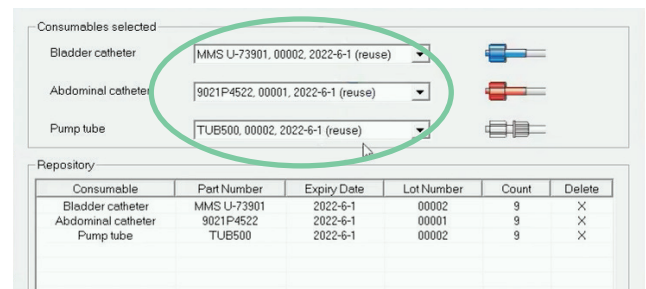


6. Multiple investigations can be performed with same consumables on same patient within the 2-hour window.

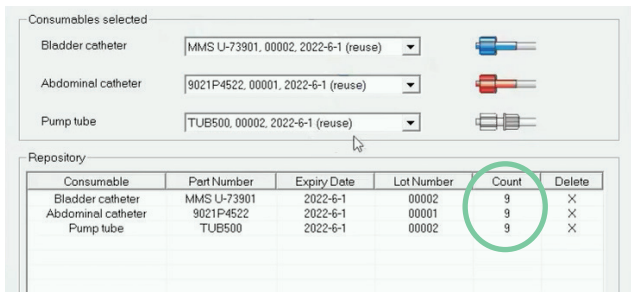


7. After the study, the consumables are deducted from the scanned inventory.

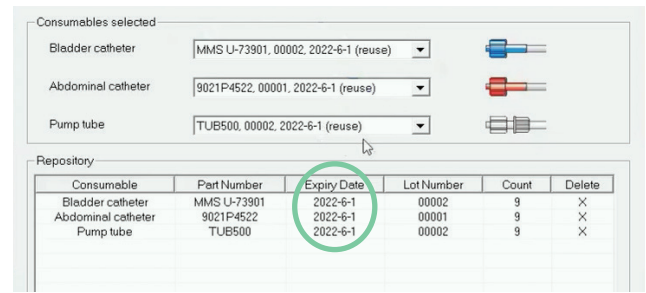


8. The system will automatically detect expired consumables and remove them from the scanned inventory. The screen shows an alert one month prior to expiration date.

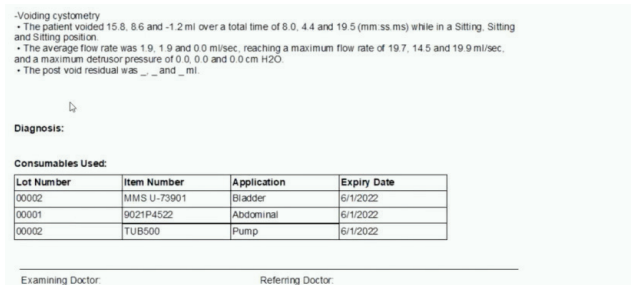


9. The used consumables appear in the uroynamics study report.

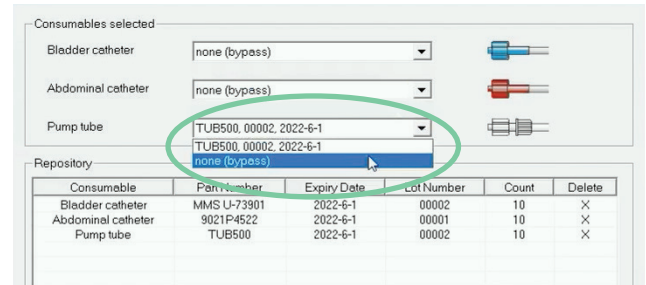

10. Bypasses allow the uroynamics study to be performed if there is a problem with the RFID tag scanning.

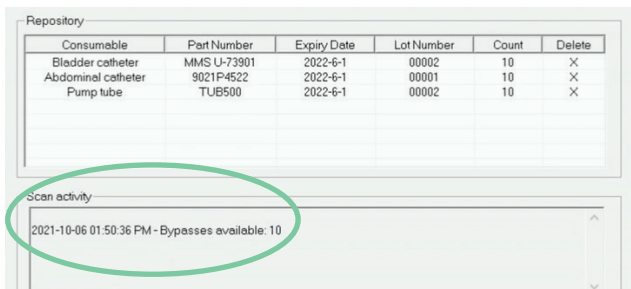

11. For every 10 studies, one bypass will be added to the bypass inventory automatically by the system.

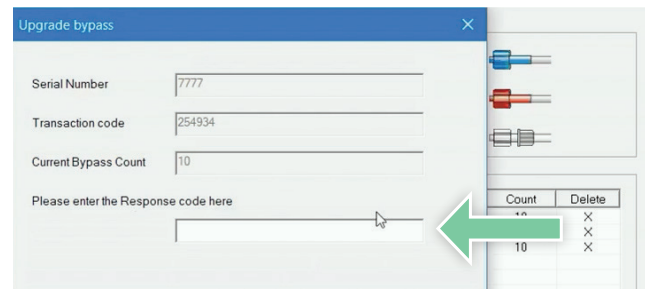

12. In the event no more bypasses are available, contact Laborie Technical Service to insert the Response Code.

Some Laborie products may not be available in certain regions. Please contact your local Laborie sales representative to learn more about the products available to you.

© 2021 Laborie. All rights reserved.

Laborie¹

USA

Pease International Tradeport T +1 802 857 1300
 180 International Drive, F +1 802 878 1122
 Portsmouth, NH 03801, USA W laborie.com



Quick User Guide Video



Learn more

MKT-00495-A