



Take back your
confidence by
taking control
of your bowels





With the
Eclipse™ System,
**control is in
your hands.**



Eclipse System is an FDA-cleared, simple-to-use vaginal insert clinically proven to reduce the frequency of bowel accidents, or even eliminate them altogether.

Sound familiar?

- ☐ **Urgency:** Sudden, strong urge to have a bowel movement
- ☐ **Accidents:** Can't make it in time
- ☐ **Passive soiling:** Leaking stool without knowing it
- ☐ **Incomplete emptying:** Remaining stool leaks out later
- ☐ **Staining** in undergarments
- ☐ **Constantly going** to the bathroom

Why does this happen to women?

Bowel control issues affect 1 out of 11 adult women.¹ The pelvic muscles that support your bladder, rectum and uterus can become damaged—often through pregnancy or childbirth. This sometimes causes loss of bowel control. Gastrointestinal conditions such as Irritable Bowel Syndrome can also lead to bowel control issues.

You may have tried or heard about these ways to regain bowel control:

- Changing diet, using medications and wearing protection.
- Performing pelvic floor physical therapy.
- Or you may opt for a surgical solution such as sphincter repair or sacral nerve stimulation.
- Or you may opt for the Eclipse System, a vaginal insert clinically proven to reduce the frequency of bowel accidents, or even eliminate them altogether.

Good news:


Eclipse can be used as a stand-alone therapy, or alongside other therapies to further enhance treatment effectiveness.





No surgery. No drugs. Just a simple solution.

The **Eclipse System** is a vaginal Insert that uses your body's anatomy to prevent accidents.

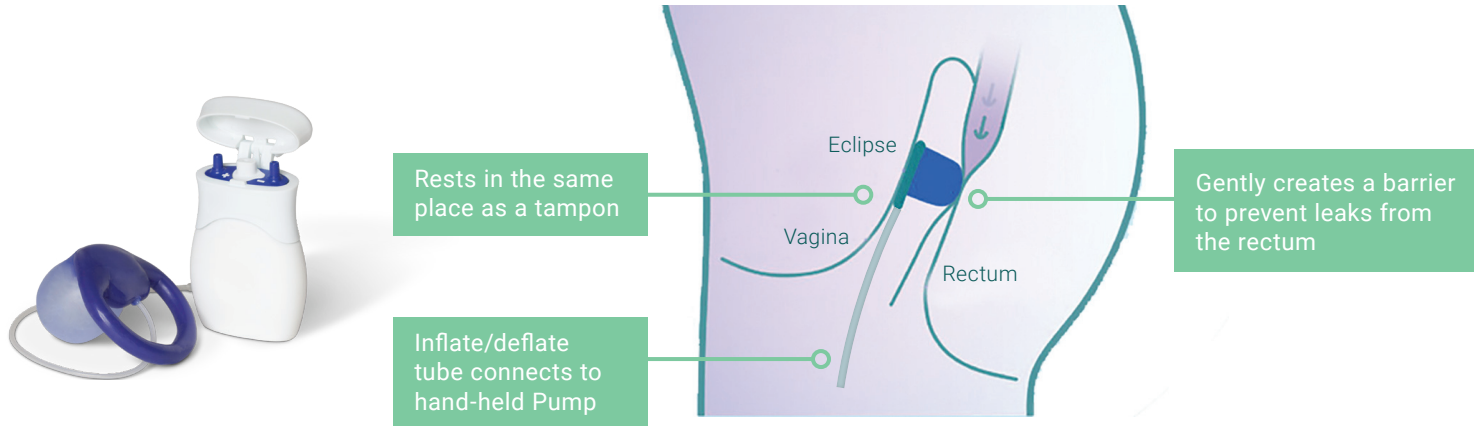

What do other women think about using Eclipse?



84% of the women said Eclipse was effective at 3 months³



96% of the women who were fitted with Eclipse said it is comfortable²



98% of the women said they would recommend it to a friend²





Just try it!



The **Eclipse System** works as soon as you put it in place.



Your doctor or medical professional fits it for you in a short office visit.



Use it when **you** want control, security, and confidence.

With **Eclipse**, **you are in complete control.**

A quick start to lasting confidence

1



First, try it. Prior to receiving your Eclipse System, you will be fit with a temporary device to make sure it is right for you.

2



Be confident. You are now ready to use your Eclipse whenever you want.

3



You're in control. Wear it every day or only when you want to feel more secure...and prevent embarrassing accidents.

Ask your doctor about the **Eclipse System** today.

Scan the QR code for more information, or

visit www.laborie.com/product/eclipse/



ECLIPSE SYSTEM IMPORTANT SAFETY INFORMATION

Indications for Use

The Eclipse System is indicated for the treatment of fecal incontinence in adult women. It is intended for prescription use.

Contraindications

The Eclipse System is contraindicated in the presence of a vaginal infection or open vaginal wound.

Warnings

The Insert should be removed before obtaining an MRI to avoid adverse events. Women should consult a doctor before obtaining a pelvic X-ray since the Insert may obscure images. The Insert and the Pump are for single-patient use only. Sharing the devices could result in transmission of disease and/or infection. Women should remove the Insert immediately and notify their doctor if they experience any of the following: a foul odor or excessive vaginal discharge, difficulty urinating or defecating, significant bleeding not associated with menstruation, or any new onset or worsening pelvic pain or discomfort.

Risks

In a clinical study of the Eclipse System, 117 women participated in the fitting process, and 61 women were successfully fit and entered a 1-month Treatment Period evaluation. Among the 61 women who entered the Treatment Period, there were no serious device-related adverse events. The most common adverse event was pelvic cramping and discomfort, and other observed events included pelvic pain, vaginal spotting and urinary incontinence. All adverse events were rated as mild or moderate, and lower rates of adverse events were observed after the initial Fitting Period was complete. All adverse events were resolved without surgery or significant intervention; most simply required removing the insert.

Please refer to the Eclipse System Instructions for Use (www.eclipsesystem.com/IFU) for a complete discussion of risks, contraindications, warnings, and precautions.

- 1 Nygaard I, Barber MD, Burgio KL, et al. Prevalence of Symptomatic Pelvic Floor Disorders in US Women. *JAMA*. 2008;300(11):1311-1316.
- 2 Richter HE, Matthews CA, Muir T, et al. A Vaginal Bowel-Control System for the Treatment of Fecal Incontinence. *Obstetrics & Gynecology*. 2015; 125(3):540-547.
- 3 Richter HE, Dunivan G, Brown H, et al. A 12-Month Clinical Durability of Effectiveness and Safety Evaluation of a Vaginal Bowel Control System for the Nonsurgical Treatment of Fecal Incontinence. *Female Pelvic Med Reconstr Surg*. 2019; 25:113-119.