

Break the anterior urethral stricture cycle

Optilume® Order Numbers

Inflated Balloon Diameter (Fr)	Content	Balloon Length (3cm)	Balloon Length (5cm)
		Product SKU	Product SKU
18.0	Optilume® Urethral Drug Coated Balloon & Inflation Device	OPTBDL7000C	OPTBDL7001C
24.0	Optilume® Urethral Drug Coated Balloon & Inflation Device	OPTBDL7002C	OPTBDL7003C
30.0	Optilume® Urethral Drug Coated Balloon & Inflation Device	OPTBDL7004C	OPTBDL7005C

1 Data on file. DSC016-004 rJ Robust I Clinical Study Final Report

2 Steenkamp JW, Heyns CF, de Kock MLS, Internal Urethrotomy Versus Dilatation as Treatment for Male Urethral Strictures: A Prospective, Randomized Comparison. Journal of Urology, Vol. 157:96-101. January 1997


3 Data on file. RP1277-001 rB

4 Data on file. TP1041




This document is for use in the European Union only.

©2024 Laborie. All rights reserved.

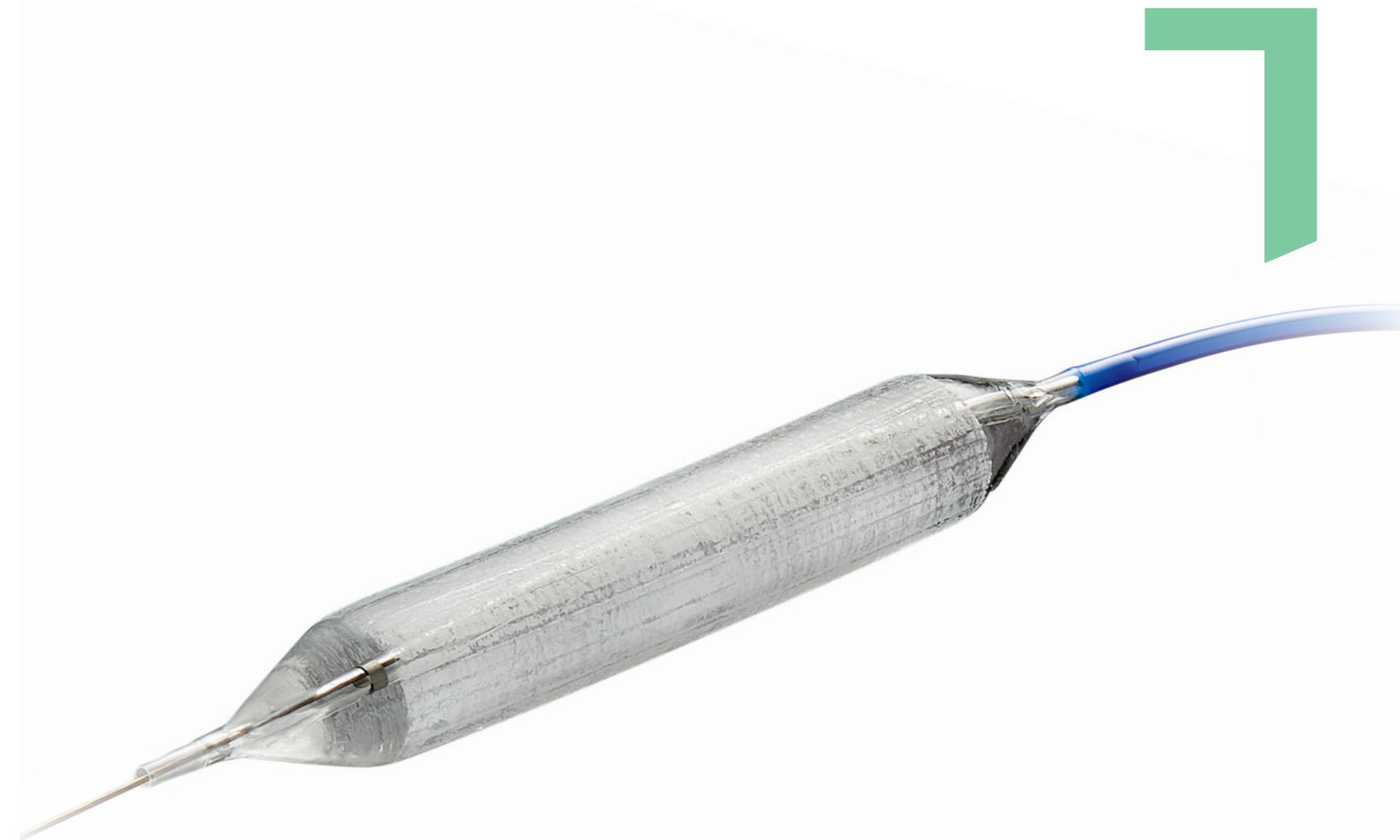

To incorporate Optilume® into your practice, please visit us at www.optilume.com

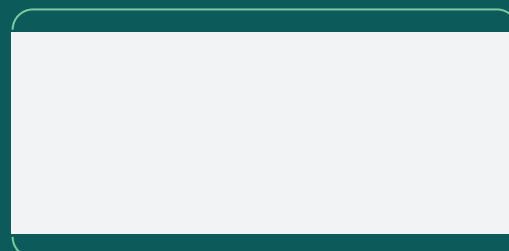


Simple. Safe. Durable.

A breakthrough treatment for anterior urethral strictures

For patients who suffer from anterior urethral strictures, today there is Optilume® – a breakthrough urethral drug coated balloon (DCB) treatment that dilates the urethral lumen and delivers paclitaxel directly to the stricture, significantly reducing the incidence of stricture.¹

Until now, there has been no treatment for anterior urethral stricture that provides ambulatory recovery, minimal complications and durable outcomes, all in a simple outpatient procedure.²


Optilume® is the cost-effective alternative¹ to endoscopic management that breaks the cycle of anterior urethral strictures.

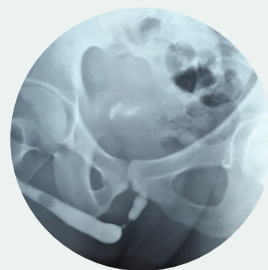

Baseline

Baseline


Length of stricture: 1.0 cm
IPSS: 29
Qmax: 4 mL/sec

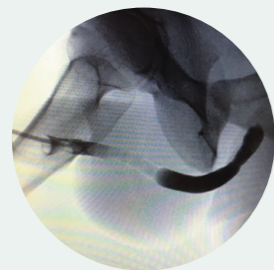

6 months

24-month follow-up

IPSS: 1
Qmax: 18 mL/sec

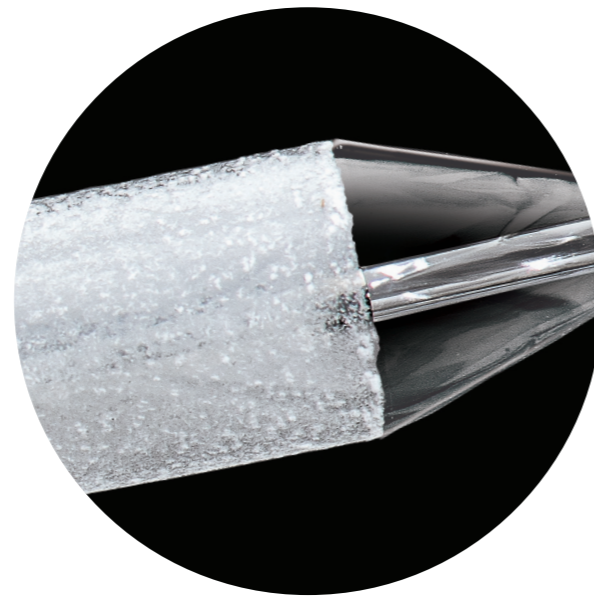


▲ Paclitaxel coating

The Optilume® Balloon Catheter has a significant opportunity to be a game changer for this disease.

– Gerald H. Jordan MD, FACS, FAAP (hon), FRCS (hon)

Simple

Easily achieve relief

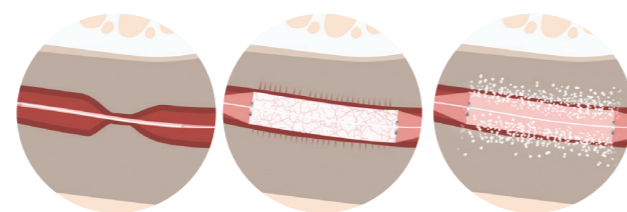

1. Insert proprietary semi-compliant balloon
2. The balloon expands the lumen creating microfissures in the tissue
3. The coating releases paclitaxel directly into the fissured tissue. Paclitaxel is absorbed by the urothelium where it resides through 28 days, reducing cell proliferation and fibrotic scar tissue generation⁴

Safe

Proven on a very difficult patient population

In the ROBUST trials with patients who had received multiple prior dilations, there were:

- No device-related serious complications¹**
- No impact on erectile function¹**





Effective

73%

Freedom from repeat intervention at 5 years³

▲ At 5 years, durability continued with 73% FFRI, with a 298% increase in Qmax and a 71% decrease in IPSS.

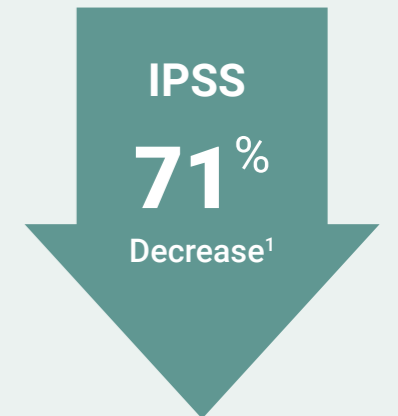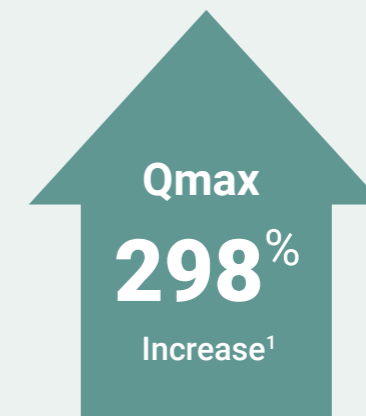

Durable

The breakthrough treatment to break the cycle

Optilume® delivers **sustained relief** due to paclitaxel's ability to limit fibrosis that leads to anterior urethral stricture¹