


traxiTM

Quick Reference Guide - traxi Retractor

OBSTETRICS, GYNECOLOGY & NEONATAL
traxi easily retracts the panniculus
for any abdominal procedure, freeing
staff to care for the patient.

1. Manual Retraction

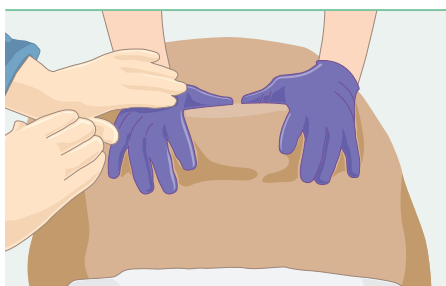


Manually retract panniculus to expose surgical site.

2. Remove 'A' Tab




Remove Tab 'A'.

3. Position & Apply

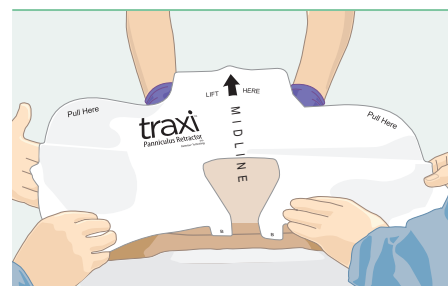


Position retractor 5cm above incision line.
Align to midline, apply to patient.

4. Remove 'B' Panels

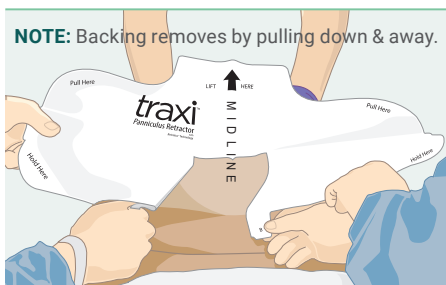


Remove the 'B' panels while holding film in tension above patient.

5. Tension & Apply

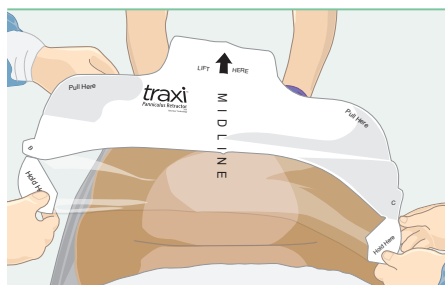


Hold in tension while smoothing down onto patient's skin, midline outward.

6. Remove 'C' Panel

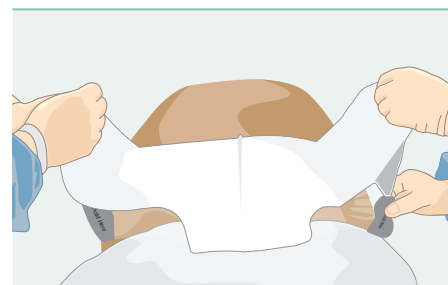


Retrograde device to relax panniculus to natural position. Remove 'C' panel by simultaneously pulling 'C' tabs.

7. Tension, Lift, Pull

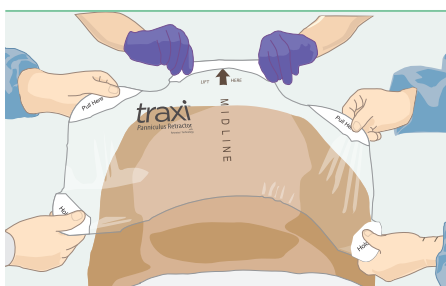


Together, in tension, lift and pull retractor cephalad, anchoring at the xiphoid.

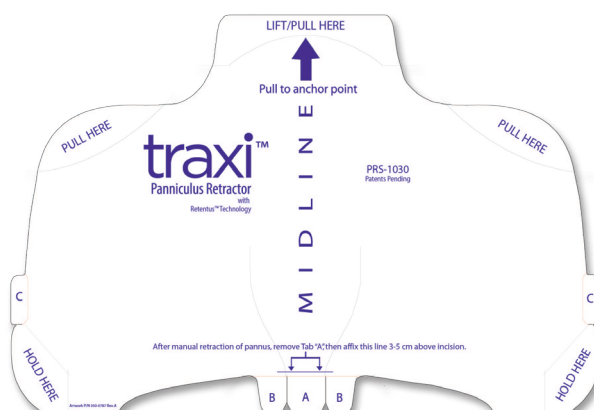


See Instructions for Use for full instructions, warnings, precautions and contraindications.


1. Apply traxi

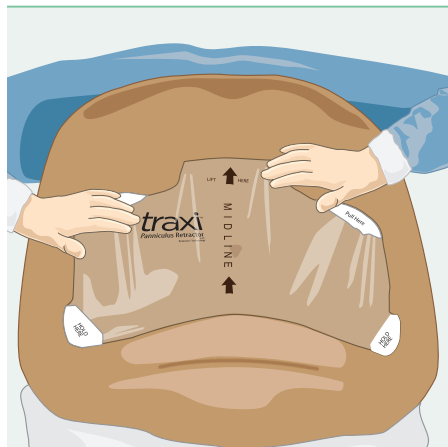

Apply traxi retractor. If suboptimal retraction is achieved, extender may be used.

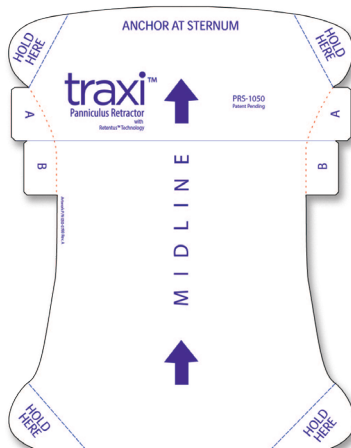


2. Expand the Anchor Point

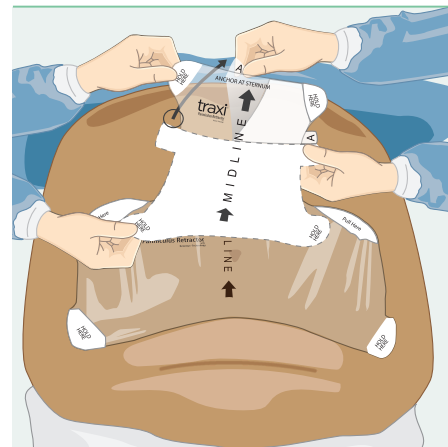


Remove 'A' panel by simultaneously pulling 'A' tabs from each side. Apply to sternum between breasts, near sternal notch midline — smooth onto skin.

3. Remove 'B' Panel

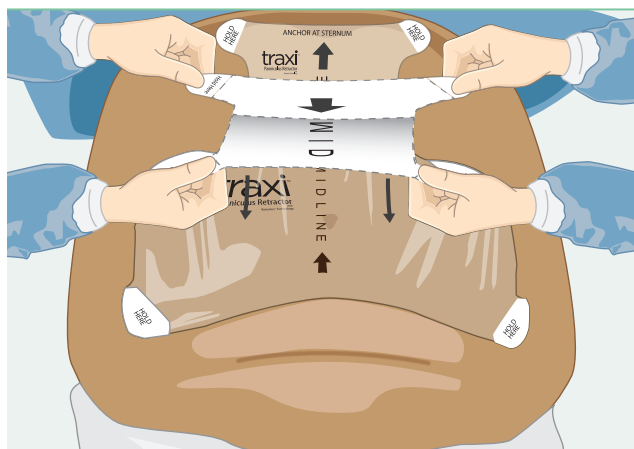

Remove 'B' panel by simultaneously pulling 'B' tabs from each side. Prepare to overlap and apply to traxi (do not apply yet).

4. Secondary Retraction

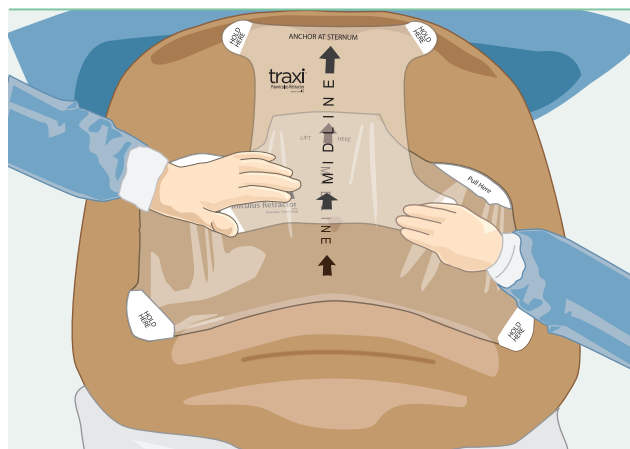

Before applying the extender, push the panniculus upward (toward patient's head). Holding the panniculus in place, smooth extender down onto the traxi toward umbilicus.

See Instructions for Use for full instructions, warnings, precautions and contraindications

© 2024 Laborie. All rights reserved.

