

BoogieBaby™

Nasal and Oral Aspirators

OBSTETRICS, GYNECOLOGY & NEONATAL

Range of single-use nasal and oral aspirators for premature, term newborn and older babies.


Your patient, our priority

The BoogieBaby™ range of nasal and oral aspirators are designed in collaboration with clinicians. They use smooth materials that are gentle on neonates and infants whilst maintaining a cost effective solution.




A Variety of Tips


The variety of tips make BoogieBaby ideal for use throughout your hospital – NICU, PICU, delivery room, emergency room – wherever suctioning is needed.

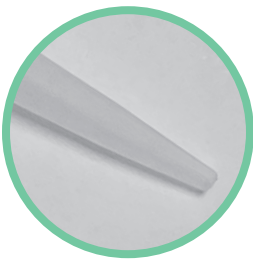


▲ **Preemie tip**
Tapered smooth tip.
For premature babies.

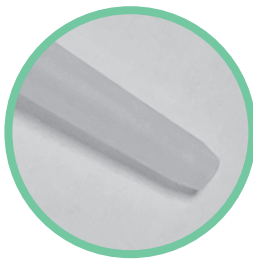

▲ **Standard tip**
Tapered smooth tip.
For term newborn and older babies.

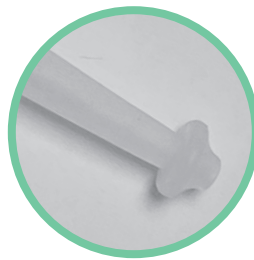

▲ **Flair tip**
Minimizes risk of trauma by creating a seal at the nares.
For all newborn babies.

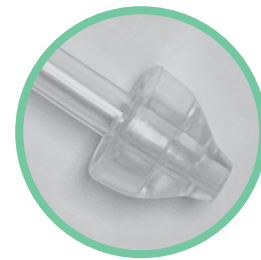

▲ **Olive tip**
Same benefits as the Flair tip.
Transparent tubing. Available in 3.25" and 4.5" length. For all newborn babies.

Single Patient Use

Packaged individually for single patient use. Should be disposed of after 24 hours.

Cover Option

Available with or without a cover to allow for intermittent suctioning. A unique removable hook allows the device to be hung at the bedside, while allowing for simple one-handed removal of the cover for quick access.

Controlled Suctioning

Ergonomically designed to facilitate single handed operation and easy access to the thumb port for controlled suctioning.


Reduced trauma

The Flair and Olive tips have a wider tip to create a seal at the nares and to help avoid trauma.

BoogieBaby™ Quick Reference Guide


1. Connect

Connect to standard suction line

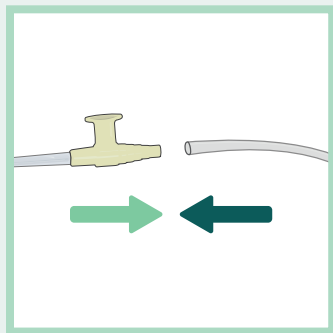

2. Set pressure

Set suction pressure and use per hospital protocol

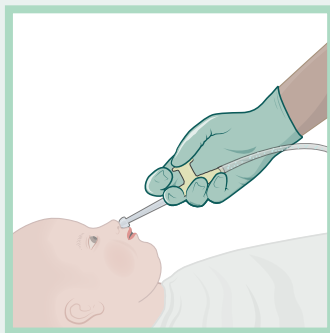


3. Clean after use

Suction sterile water through device after each use



4. Replace

Replace at least every 24 hours. Dispose of the device per hospital protocol.

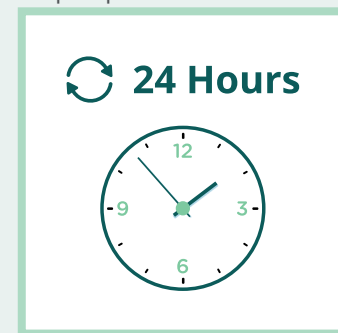

Always follow hospital policy and procedures for further guidance.

See Instructions for Use for full instructions, warnings, precautions and contraindications.

Ordering information

Part Number	Description	Units/Box
VPBB15011100	Preemie tip	100
VPBB15012100	Preemie tip with cover	100
VPBB15013100	Standard tip	100
VPBB15014100	Standard tip with cover	100
VPBB15015100	Flair tip (for nasal use only)	100
VPBB15016100	Flair tip with cover (for nasal use only)	100
VPBB150112100	Olive tip 4.5 inch long (for nasal use only)	100
VPBB150113100	Olive tip 3.25 inch long (for nasal use only)	100
VPBB150114100	Olive tip 4.5 inch with cover (for nasal use only)	100
VPBB150115100	Olive tip 3.25 inch with cover (for nasal use only)	100

© 2022 Laborie. All rights reserved.

Laborie¹

USA

T +1 800 522 6743
F +1 802 878 1122

E usmarketing@laborie.com
W laborie.com



Learn more

ART-0319 Rev 02