


ROM Plus[®] ES

Quick Reference Guide


OBSTETRICS, GYNECOLOGY & NEONATAL
Accurately testing for proteins found in cervicovaginal secretions to help diagnose ruptured fetal membranes.

1. Prepare Test

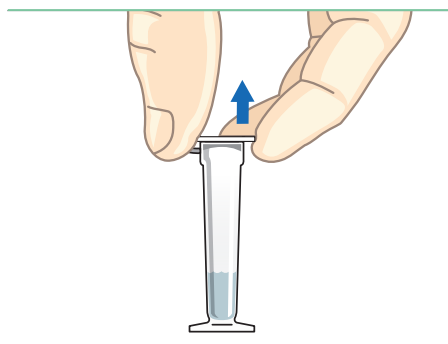


Remove ROM Plus contents from packaging. Holding buffer vial in upright position, remove shipping cap and set it aside.

2. Collect Sample

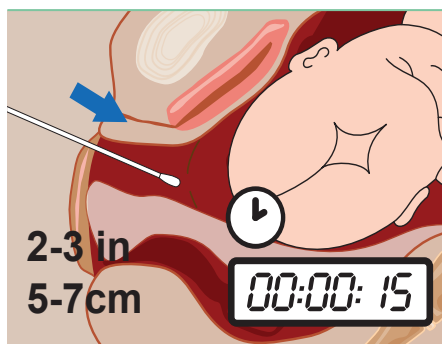


Remove sterile swab from package. Insert swab tip into vagina 2-3 inches (5-7 cm). Withdraw swab after a minimum of 15 seconds.

3. Mix Sample & Buffer

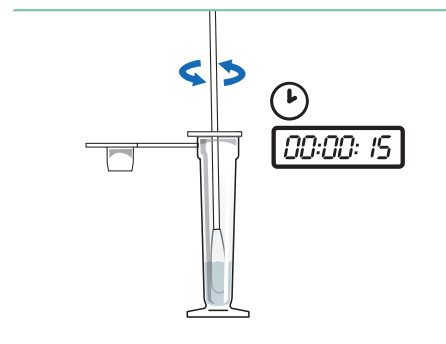


Place the swab tip into the vial. Mix the swab in the buffer for at least 15 seconds, then remove and dispose of the swab.

4. Insert Test Strip

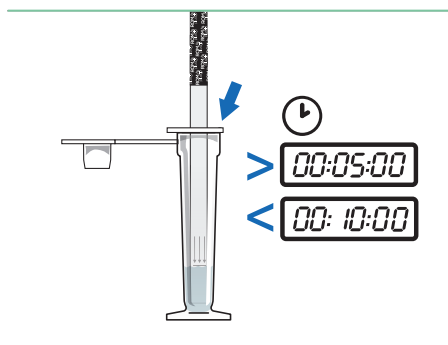

Remove ROM Plus test strip from foil pouch. Place white end of strip into vial with buffer solution.

5. Remove & Interpret

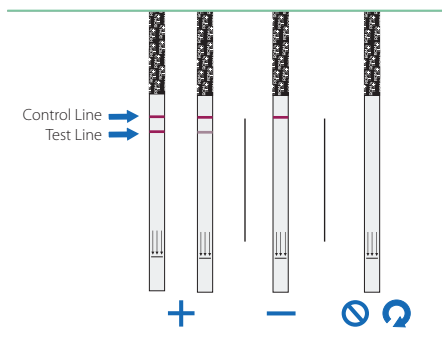

Remove test strip if two lines are clearly visible or after 10 minutes. The test is valid even if lines are faint. If only a control line (top line) is visible, the test result is negative. If two lines (control line and test line are visible) the test result is positive. If the control line is not present, or if no lines are visible, the test result is invalid and should be repeated.

See Package Insert for Full Instructions, Warnings, Precautions & Contraindications

© 2024 Laborie. All rights reserved.

